# Mediblink®

medical devices

## **EN** Mediblink Electronic nasal aspirator M420

### INSTRUCTIONS FOR USE

Please read this guidebook completely before operating this unit.

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### UPUTE ZA UPOTREBU

Pažljivo pročitajte upute prije upotrebe uređaja

Introduction	6
Important Instructions Before Use	7
Components	11
Inserting Batteries	12
Music Function: 12 World-known Ballads for Children	15
Use of the Product	15
Care and Maintenance	17
Errors and Troubleshooting	19
Technical Specifications	20
Electromagnetic Resistance to Interference – EMC Tables	21
Warranty Conditions	24

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### 1. Music function

You can choose between 12 different melodies.

### 2. Powerful motor

Insert the nasal nozzle of the aspirator into a nostril and remove the liquid/solid nasal discharge.

### 3. Safe for use

The soft silicone nasal nozzles are specifically designed to prevent any injury during the use of the product.

### 4. Compact design, easy to use

Simple to hold and use. It is safe and comfortable to use for babies.

### 5. Detachable and washable aspirator head

The nozzle, the container for nasal discharge, and the aspirator head can be removed from the aspirator and washed. The container for nasal discharge can be disinfected in boiling water.

### 6. Low weight and low noise

It's easy to use even while the baby is sleeping.

### 7. Battery

Two 1.5V AA batteries are included. You can easily replace your batteries with new ones: anywhere and anytime.

## Important Instructions for Use of the Product

### **Warning! Potential risks for children and people with reduced physical, sensory, or mental abilities!**

1. This aspirator is not intended to be used by persons (including children) with limited physical, sensory or mental abilities, or lack of experience and/or knowledge, unless they are supervised by a person responsible for their safety, or they have received instructions from such person on how to use the aspirator.
2. Keep the aspirator out of the reach of children under 18 years of age.
3. Supervise your children all the time so they don't play with the aspirator.
4. Children are not allowed to handle the aspirator. They could swallow small particles or batteries and suffocate. Children could suffer injuries if they use the aspirator.

### **Warning! Risk of personal injury!**

1. Improper handling of the aspirator may result in personal injury.
2. Do not use the aspirator in case of malfunction. Do not attempt to modify, disassemble or repair the aspirator, and do not replace its parts.
3. If abnormalities occur during the use of the aspirator, stop the use immediately and consult your physician.
4. Do not perform any other activities during the suction of nasal discharge. Be still.
5. If you have any doubts regarding the use of the aspirator, please consult your physician.
6. These instructions for use must be stored for the duration of the product life cycle and handed over to a third party along with the aspirator. The instructions for use should also be accessible to third parties. The instructions for use are an integral part of the aspirator.
7. Avoid improper use.

## Important Instructions for Use of the Product

8. This aspirator cannot replace medical treatment! In the event of a medical problem, you should see your physician.
9. The nasal nozzles, the container for nasal discharge, and other components of the aspirator must be clean and dry. Avoid damaging the aspirator. Only this way you can guarantee it will operate efficiently.
10. If the components of the aspirator are dirty, the suctioning may be less efficient.
11. Do not use the aspirator if it is damaged. Using a damaged device may cause personal injuries or serious danger.
12. While using the product, the baby should not hold its breath, close its mouth, or swallow saliva, in order to avoid the imbalance of the eardrum, caused by the vacuum.
13. If your baby does not feel well during the process, stop using the device immediately and consult your physician.
14. Do not force the silicone nozzle into the nasal cavity. Insert the nasal nozzle gently.
15. The product is intended for domestic use only. Consult your physician in case of any nasal disease or disease related to nasal congestion. Before each use, rinse the silicone nasal nozzle.
16. **DO NOT USE the aspirator when the patient has the following contraindications:**
  - nosebleed
  - acute head, face, or neck injury
  - blood clotting disorders (coagulopathy)
17. Before using the product, please read the following instructions carefully. Improper use of the product may result in the following risks or complications:

## Important Instructions for Use of the Product

Mechanical damage of the product  
Nasal irritation/nosebleeds  
Nasal mucosa swelling  
Discomfort and pain

18. Please keep silicone nasal nozzles out of the reach of children and animals so they cannot swallow them.
19. The nozzle tip may cause allergic reactions. Consult your physician before use.
20. The device is activated, creates a vacuum, and allows the suction of nasal discharge only when you press and hold the ON button. As soon as the button is released, the device automatically turns off.

**Warning! Risk of injury!** Improper handling of the aspirator may cause injuries.

1. Do not drop the aspirator on the floor, shake it, or expose it to impacts.
2. To ensure effective suctioning of the nasal discharge, the product must be protected from moisture, dirt, and damages.
3. Do not expose the aspirator to direct sunlight or to extremely high or low temperatures. The aspirator must be used and stored only in a dry and clean environment.
4. Do not open the aspirator and do not attempt to fix it yourself if malfunction or damage occurs. Doing so would void the warranty claim. Only authorised persons are allowed to repair the aspirator.
5. Do not store the aspirator in a place with extreme temperatures (below  $-20^{\circ}\text{C}$  or over  $55^{\circ}\text{C}$ ), or in an extremely dry or moist place, as this could cause the device to stop operating.


## Important Instructions for Use of the Product

6. Do not use the aspirator in a wet environment. While using the aspirator, keep it away from liquids and never immerse it in liquid.
7. Do not place the aspirator near flammable materials and gases or near explosives.
8. Do not use the aspirator in the mountains at an altitude of more than 3,000 metres.
9. Electrical medical devices are subject to specific safety regulations concerning electromagnetic compatibility. You must comply with these rules when installing and using the aspirator.
10. Only use the aspirator strictly for the purpose of use stated in these instructions for use. Do not use the device for any other purposes.

The Mediblink M420 nasal aspirator can only be used with the original Mediblink M420 nozzles. Please contact your dealer or distributor for replacement parts.

## Components of the Product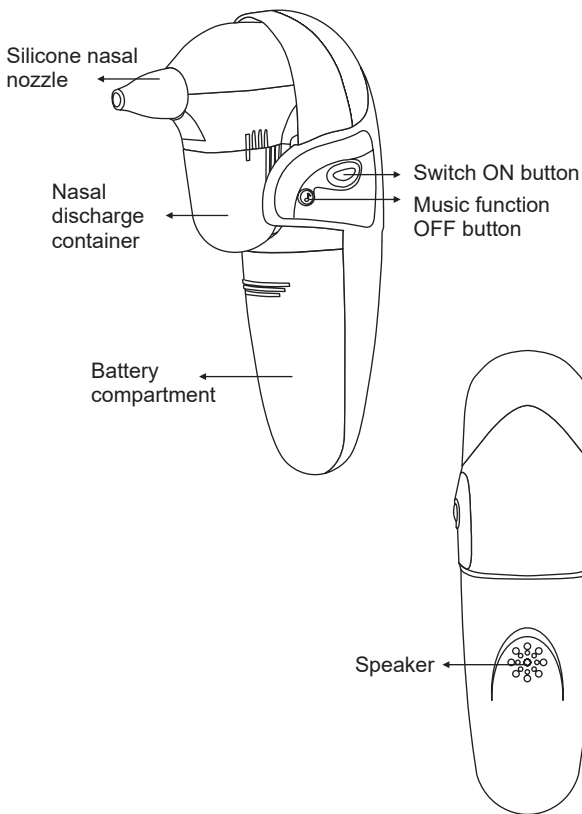

## Inserting Batteries

### **Warning! Risk of explosion!**

Improper handling of batteries may result in explosion.

1. Only replace batteries with equal batteries or batteries of an equivalent type.
2. Do not expose batteries to excessive heat such as the sun or fire.
3. Do not charge, re-activate, disassemble, or throw the batteries into fire, or cause a short-circuit.

### **Warning! Risk of personal injury!**

Improper handling of batteries may result in injuries.

1. Keep the batteries out of reach of children.
2. Swallowing the batteries can be life-threatening. Therefore, store the batteries out of reach and sight of children. If a battery is swallowed, seek medical advice immediately.
3. If battery fluid leaks, avoid contact with skin, eyes, and mucous membranes. In case of contact with acid, wash the affected area immediately with plenty of clean water and immediately see your physician.

### **Warning! Risk of injury!**

Improper handling of batteries may result in injuries.

1. When inserting the battery, make sure that the terminals are properly oriented.
2. Only insert batteries of the same type in the battery compartment. Do not use batteries of different types or used and new batteries together.
3. Remove the old batteries from the battery compartment if they are empty, or if the aspirator will not be used for a longer period.


## Inserting Batteries

### Low battery

Before the batteries are completely empty, the power of suctioning will be reduced and/or the music will no longer play. In that case, change the batteries.

### Changing the batteries

1. Press and slide gently to remove the battery compartment cover on the back of the product.
2. Carefully remove the old batteries.
3. Insert two new 1.5V AA batteries into the battery compartment. When inserting the batteries, make sure that the terminals are properly oriented (the + and – terminals on the batteries and on the bottom of the battery compartment must match). Do not use rechargeable batteries!
4. Slide the battery compartment cover back into place.

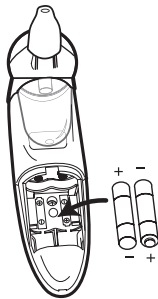## Disposal

### Disposal of packaging

For disposal, separate the packaging by the type of material. Handle paperboard and cardboard as waste paper, and the foil as secondary raw materials.

### Disposal of waste product

(Applicable to the European Union and other European countries with systems for separate collection of secondary raw materials.)

### Waste devices may not be disposed together with household waste!

When the aspirator is no longer functioning, the consumer is **legally required to dispose of the old product separately from household waste**, e.g. at the collection point of their municipality/area where they reside. This will ensure that the old devices are professionally recycled and that negative environmental impacts are prevented. Therefore, electrical devices are marked with the symbol shown here.

Put the aspirator in waste electrical and electronic equipment in accordance with EC Directive 2012/19/EU.

### Batteries and rechargeable batteries may not be disposed together with household waste!


As a consumer, you are required by law to dispose of all batteries and rechargeable batteries, whether or not they contain harmful substances\*, at a collection point in your municipality/area or at a store, in an environmentally friendly manner.

\* with the following elements: Cd = cadmium; Hg = mercury; Pb = lead

## Music Function: 12 World-known Ballads for Children

If you want the device to play music that distracts the child during the suctioning, press the Music ON button.

1. Instead of listening to the dull sound of the aspirator, our product gives you a chance to choose from 12 world-known ballads.
2. Press the "Music ON" button to activate the music function. By continuing to press this button you can alternate between different melodies.
3. When you stop using the product, press and hold the "Music ON" button for 1 second to turn off the music.

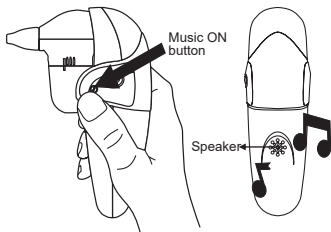## Use of the Product

1. Select the correct nasal nozzle considering the shape and size of the baby's nasal cavities.

### Large nasal nozzle


The large nozzle is used when discharge is in solid form.

### Narrow nasal nozzle


The narrow nozzle is used when discharge is in liquid form.

2. Make sure that you attach the nozzle properly on the aspirator. Before use, make sure that the nasal nozzle covers the ribbed part of the aspirator tip.

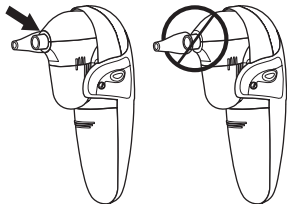
## Use of the Product

3. To start the suctioning of discharge, press the “ON button”.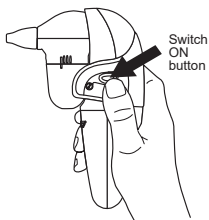

4. Hold the baby in an upright position and insert the aspirator nozzle into the nasal cavity of the baby's nose. Make sure not to insert the nasal nozzle too far up the nasal cavity, and not to scratch it on the inside. Make sure you hold the aspirator at 90° (as shown in the figure). Suctioning will remove mucus from the nose. If necessary, repeat the procedure on the other nasal cavity.

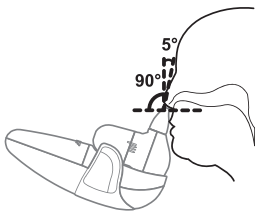-  The mucus will not necessarily appear in the nasal discharge container immediately. After each use, remove the nasal nozzle from the aspirator and clean it.



5. Release the aspirator ON button to stop the suctioning.

## Care and Maintenance


For the product to remain efficient, it is very important that it is kept clean.

1. Separate the nozzle and the aspirator head from the aspirator by gently pulling the components apart.
2. Unscrew the nasal discharge container from the aspirator head clockwise.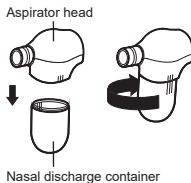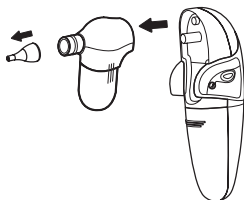

### Cleaning the discharge container, the aspirator head, and nasal nozzles:


Wash the discharge container, the aspirator head, and nasal nozzles with mild soap for 3 minutes. Then rinse all of these components under hot running water for 3 minutes. After cleaning all elements, disinfect them in boiling water.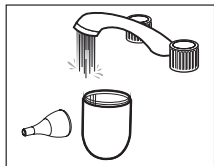

**i** **Do NOT wash with detergents!** Please wash the components only with running water.

### Cleaning the aspirator

Wipe the device with a clean, gentle, and dry cloth.

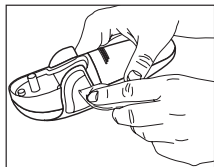## Care and Maintenance

**Do NOT wash the device with water or any other fluid!** Use a dry, soft cloth for cleaning the housing. After cleaning the aspirator, store it in a clean, dry place at room temperature. Never expose the aspirator to extreme temperatures, humidity, direct sunlight, or shaking! Do not rinse the aspirator under water!

### **Warning! Risk of short circuit!**

If water enters the housing of the product, a short circuit may occur.

1. Never immerse the aspirator in water.
2. Make sure that water does not enter the housing.

### **Warning! Risk of injury!**

Improper handling of the aspirator may cause injuries.

1. Do not use aggressive cleaning agents, brushes with metal or nylon bristles, or sharp or metallic cleaning items such as knives, hard spatulas, or anything similar. They can damage the surface of the device.
2. Never clean the aspirator with a polishing agent, gasoline, or solvent.

## Errors and Troubleshooting

Problem	Cause	Solution
Weak suctioning or no suctioning.	Batteries are empty or incorrectly inserted.	Check or change the batteries.
	The gray rubber ring is missing (it must be placed in the area where the discharge container is wound up on the aspirator head).	Make sure that the gray rubber ring is fitted on the threaded area where the discharge container is to be installed on the aspirator head.
	The discharge container is not properly installed.	Make sure that the 3 lines on the nasal discharge container match the lines on the part of the aspirator that allows suctioning.
No music when you press the Music Function ON button.	The music is still playing.	The music will automatically stop when the selected song ends. By pressing the Music Function ON button you will not stop the music.

## Technical Specifications

**Product:** Mediblink Nasal Aspirator M420, model: NS12

**Motor and vacuum power:** 150mmHg (60kPa)

**Motor:** 3V/DC

**Volume:** 75–80dB

**Suction nozzle type:** 2 soft silicone nozzles

**Conditions for use:** 16°C~35°C (60.8°F~95°F), with relative humidity of up to 85% (not liquefied)

**Storage and transport conditions:** -20°C~+55°C (-13°F~+131°F) with relative humidity of up to 85% (not liquefied)

**Power supply:** 2×1.5V AA alkaline batteries

**Estimated battery operation time:** approximately 90 minutes (time of operation depends highly on the battery manufacturer)

**Weight:** approximately 146g (excluding batteries)

**Product dimensions:** approximately 98.5mm×45.30mm×143mm (L×W×H)

**Estimated time until servicing:** 5 years

	Warning!
	Before use, read the instructions. Electronic instructions for use: <a href="http://www.mediblink.com/f/m420.pdf">http://www.mediblink.com/f/m420.pdf</a>
	
	Product reference number
	Class IIa medical device
RoHS	Skladno z RoHS direktivo 2011/65/EU
	Upon the cessation of the life cycle of the device, dispose of the device in accordance with the local law.

	LOT number*
	Product certified in accordance with EU Directive 93/42/EEC
	Serial number
	Number of products in one packaging
	Manufacturer
<b>IP 22</b>	The device is rated IP 22, which means it is protected against water and dust particles entering the device.
	EU Representative

\*Date of production: the first two digits of the LOT number represent the month of production, and the second two digits represent the year of production. Example: LOT 10/19 = October 2019.

## EMC Tables

## Guidance and manufacturer's declaration-electromagnetic emissions

The **NS12 Nasal Aspirator** is intended for use in the electromagnetic environment specified below. The customer or the user of the **NS12 Nasal Aspirator** should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment guidance
RF emissions CISPR 11	Group 1	The <b>NS12 Nasal Aspirator</b> uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The <b>NS12 Nasal Aspirator</b> is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

## Guidance and manufacturer's declaration-electromagnetic immunity

The **NS12 Nasal Aspirator** is intended for use in the electromagnetic environment specified below. The customer or the user of the **NS12 Nasal Aspirator** should assure that it is used in such an environment.

Immunity test	IEC 60601 -test level	Compliance level	Electromagnetic environment- guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV touch ± 8 kV air	± 6 kV touch ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

## EMC Tables

## Guidance and manufacturer's declaration-electromagnetic immunity

The **NS12 Nasal Aspirator** is intended for use in the electromagnetic environment specified below. The customer or the user of the **NS12 Nasal Aspirator** should assure that it is used in such an environment.

Immunity test	IEC 60601 – test level	Compliance level	Electromagnetic environment-guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the NS12 Nasal Aspirator, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance</b></p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \text{ 80 MHz to 800 MHz}$ $d = 2.3 \sqrt{P} \text{ 800 MHz to 2,5 GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey<sup>a</sup>, should be less than the compliance level in each frequency range.<sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>

**NOTE 1** At 80 MHz and 800 MHz, the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the NS12 Nasal Aspirator is used exceeds the applicable RF compliance level above, the NS12 Nasal Aspirator should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the NS12 Nasal Aspirator.
- Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

## EMC Tables

### Recommended separation distances between portable and mobile RF communications equipment and the NS12 Nasal Aspirator

The NS12 Nasal Aspirator is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the NS12 Nasal Aspirator can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the NS12 Nasal Aspirator as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 1.2 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## Warranty

**Product: Mediblink Electronic Nasal Aspirator M420**

**Manufactured for (also EU importer):** Mediblink d.o.o., Gubčeva cesta 19, 8210 Trebnje, Slovenia; info@mediblink.com; www.mediblink.com

**Sellers name, address, signature and stamp\*:**

**Date of extradition/sales\*:**

\*If the invoice is accompanied by this warranty, and if all above information can be seen from the invoice, it is not necessary to fill in this field.

### WARRANTY TERMS

Dear customers!

The warranty period is **2 years** and starts from the day of product purchase. In case of product claim, you have to show the invoice. We kindly ask you to save the invoice!

Unfortunately, wrong handling with the device is a reason for 95% of customer complains. You can easily avoid any problem, by getting useful information provided by our special service department. To reach our service department, you can call or send e-mail to Mediblink local distributor.

Before sending the product back to retailer, we kindly ask you to call our service department, to get help about how to use the device to save you with unneeded trips.

The manufacturer guarantees free elimination of all imperfections due to defects in material or manufacturing procedure by repairing or replacing the product. In case that the product can not be repaired or replaced, the customer will get the money refund. The guarantee is not valid in case of the force majeure, accidents or unexpected events (such as lightning, water, fire etc.), incorrect use or incorrect transport, non-compliance with safety and maintaining regulations or in case of unprofessional product intervention.

Traces of every day product usage (scratches, abrasions) and not subject to claim. The warranty does not eliminate the customer rights, which originate from seller responsibility for product flaws. By accepting the claimed product by the service department, the service department does not take responsibility for loss of saved data or settings on the product. All product repairs, which are performed out of product warranty period, have to be paid by customer by prior notice.

## Warranty

The manufacturer guarantees the product quality and flawless product operation in the warranty period, which starts with the day of product purchase. If the product can not be repaired in 45 days, the product will be replaced with a new one. In case that the product can not be replaced, the money will be refunded to the customer.

In case of product claim, call or send e-mail to Mediblink local distributor.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer (AViTA (Wujiang) CO., LTD), company Mediblink d.o.o. and the Competent Authority of the Member State in which the user and/or patient is established.

**Manufacturer:**  
AViTA (Wujiang) CO., LTD  
No.858. JiaoTong Road  
Wujiang Economic Development Zone  
JiangSu, P.R.C.

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